


# ENT SCOPE

## Start-Up and Operation Procedure

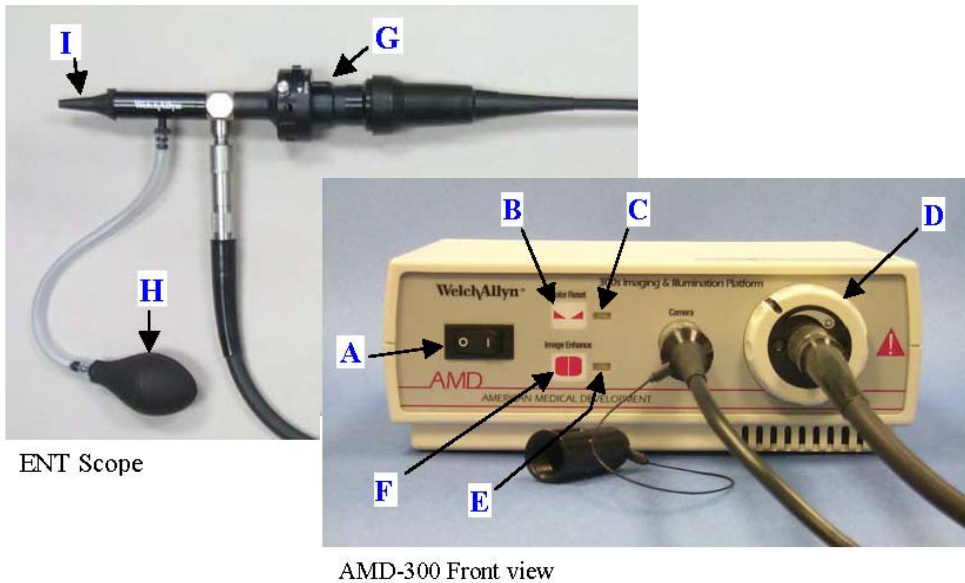

ENT Scope

AMD-300 Front view

### Start-up Procedure

1. Turn **Power switch** to "ON" (A) and look to see a light coming from the tip of the scope.
2. **Wait for about 30 sec** for SOLERC light to warm-up.
3. Turn the **Intensity Ring** (D) clock-wise to maximum illumination.
4. Point the ENT scope at a blank sheet of white paper and hold approximately 1.25 cm (1/2 inches) away. Press and hold the **Color Reset** button (B) until the light (C) stops flashing. (NOTE: This happens fairly quickly).
5. Press the **Image Enhance** button (F). Light (E) should turn on.
6. Select the **'ENT' video source on your switchbox** and then select the correct camera source on your remote control.

Note: It is recommended to show the imager from the Patient Camera as the large picture on your screen, not as the Picture-in-Picture (PIP)

### Operating Procedure

7. Point the ENT scope at any surface 1.25 cm (1/2 inch) away. Use the **Focus Ring** (G) to adjust the focus ring of the ENT scope.
8. Place a new **disposable specula** (I) for every new exam.
9. During the exam, adjust the illumination by turning the **Intensity Ring** (D) until best results.
10. After the exam, dispose of specula, clean equipment and store leaving equipment assembled.
11. Use the **Pneumatic Bulb** (H) for pneumatic otoscopy or when ENT tip gets foggy.

Note: Typically used for nasal exams and on request for pneumatic otoscopy.