

Using the Camera with OTNhub Videoconference



- 1 **Connect the lens to the camera** device. Ensure that the black dots match up and turn in a clockwise direction.
- 2 **Turn on the camera** by pressing the power button located at the top of the scope device.
- 3 **Tap the camera icon** located in the top-left of the screen and then the Settings button (⚙️). Tap on the UVC button at the top right of the screen and ensure it is on.
- 4 **Connect the USB cable** to the computer.
- 5 **Log in at otnhub.ca**, go the Videoconference service and connect to the desired system.
- 6 On the videoconference toolbar, click the **Settings** icon (⚙️), and select the appropriate peripheral camera.
- 7 **Position the camera** and use the OK button to capture and freeze the desired image. Press the OK button halfway down to enable auto focus. To return to the live camera feed, press the OK button.
- 8 To return to the room view, in the video window's toolbar, click the **Settings** icon (⚙️) and click on the main camera.

For further assistance and technical support, contact OTN Technical Support:

Phone: 1-855-654-0888 (option 2) Email: techsupport@otn.ca



Features & Controls

! The buttons and screen are active only when a lens is attached.



Power Button
(20-30 second start up)



Mini USB Port

Strap Hook



Memory Card Slot

For further assistance and technical support, contact OTN Technical Support:

Phone: 1-855-654-0888 (option 2) Email: techsupport@otn.ca

