


# ENT SCOPE

## Light Source

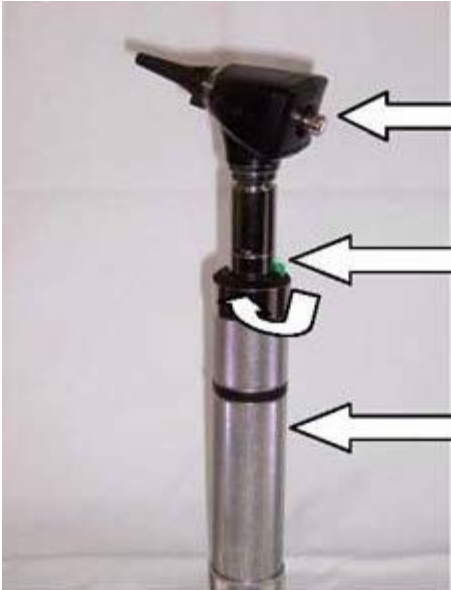

Attach video cable to the video head.

Press down on green button and rotate clockwise to turn on the light source.

Battery handle.


## Disposable Specula Placement



Only attach 4mm disposable specula to the video head.

Attach the cable to the video head.

## Recharging Otoscope System

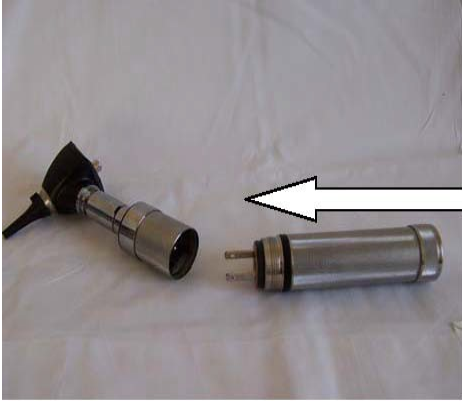

Unscrew handle to expose power outlet prongs.

Insert prongs into wall outlet to charge system.

## Rechargeable Batteries



Unscrew bottom of handle and remove batteries if they are not functioning. Otoscope requires 2 D batteries.

Note: Batteries must be kept charged as there are no other.

## Camera Source Selection



Switch from main camera to the switchbox located on right side of iDoc system.

Then, press on the second square button labeled "ENT SCOPE".