


AMD-2500 GENERAL EXAMINATION CAMERA

Start-Up and Operation Procedure

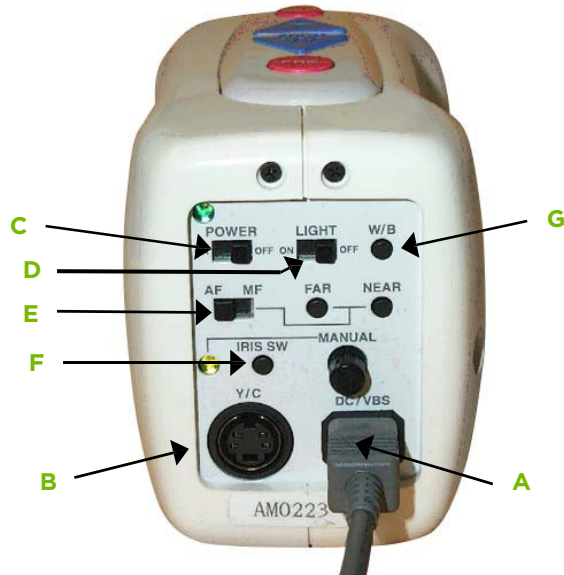

AMD-2500 Back view



AMD-2500 Top view

Start-up Procedure

1. Plug in **Grey DC/VBS** cord (A).
2. Plug in **Black Y/C** cable (B). (Note: Some systems do not require the black v/c cable due to wiring through a switcher box)
3. Turn **POWER** switch to "ON" (C).
4. Turn **LIGHT** switch to "ON" (D).
5. Set **AF/MF** switch (E) to "AF" (autofocus). (Note: Recommended setting: Autofocus)
6. Be sure that **IRIS SW** light (F) is "OFF".
7. Point the camera at a blank sheet of white paper and hold approximately 10 cm away. Press and hold the **W/B (white balance)** button (J) for 3 seconds then release. (Note: turn camera light on for white balance)
8. Zoom all the way out by holding down the blue **W** (wide angle) button (J).

Operating Procedure

9. To better view surface structures, press the red **REF** (reflection) button (H) to turn the light polarization "OFF".
10. Zoom in at small increments by pressing the blue **T** (telephoto) button (I).
11. Press **W** button (J) to zoom out to view the image at sufficient distance to show a good overall view.
12. Freeze the image by pressing the red **FRE** button (K). To unfreeze press **FRE** button again.