


## Quick Reference Guide – SSL VPN PC – Consultant site






1. Plug the Bluetooth dongle into the USB port on the computer.
  - When running properly, a green light on the USB dongle will flash.
2. To connect to the network, click on the AnyConnect (🌐) icon on your desktop and log in.
3. Turn on the Telesteth Littman 3200 stethoscope by pressing the Power button.
4. Double click on the Telesteth Icon to launch Telesteth software.
  - A Java popup and a download message appear briefly as the program starts up.
5. Enter your User Name and Password.
6. Follow the instructions to connect your stethoscope with your computer.
  - Click on the My Stethoscope down arrow, pick your stethoscope from the list and follow the instructions on the pop up window.
  - You will see a green checkmark when connected.

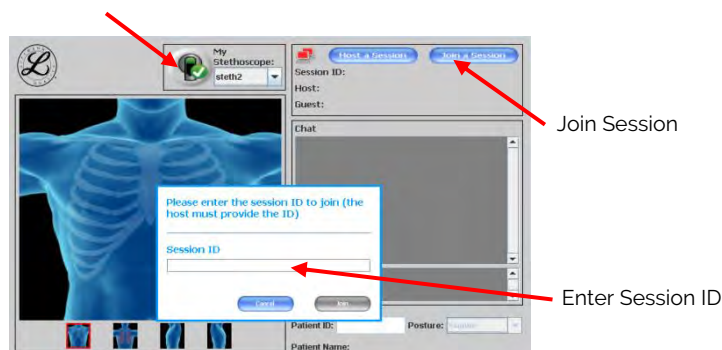

7. The Patient site will always Host a Session; the Consultant site clicks on 'Join a Session' & enters in the Session ID provided by the Patient site.
8. Once successfully connected the computer icons will turn green, and you are ready to provide placement direction to the patient site as required, adjusting the filter settings and amplification using the button on the stethoscope.

### Digital Filter Settings

Press the Filter Button until desired mode appears on screen

		<b>Diaphragm mode</b> Amplifies sounds from 20 - 2000Hz, but emphasizes the sounds between 100 - 500Hz
		<b>Bellmode</b> Amplifies sounds from 20 - 1000Hz, but emphasizes lower frequency sounds between 20 - 200Hz
		<b>Extended Range</b> Amplifies sounds from 20 - 2000Hz but provides more low frequency response between 50 - 500Hz.





9. When patient assessment is complete, click on End Session and log out of the software
10. Disconnect by right clicking the AnyConnect icon (🔒) in the taskbar and select Disconnect from the pop-up menu.

## Quick Reference Guide – SSL VPN PC – Patient site

1. Plug the Bluetooth dongle into the USB port on the computer.
  - When running properly, a green light on the USB dongle will flash.
2. To connect to the network, click on the AnyConnect (🌐) icon on your desktop and log in.
3. Turn on the Telesteth Littman3200 stethoscope by pressing the power button.
4. Double click on the Telesteth Icon to launch Telesteth software.



- A Java popup and a download message appear briefly as the program starts up.
5. Enter your User Name and Password.
  6. Follow the instructions to connect your stethoscope with your computer.
    - Click on the My Stethoscope down arrow, pick your stethoscope from the list and follow the instructions on the pop up window.
    - You will see a green checkmark when connected.

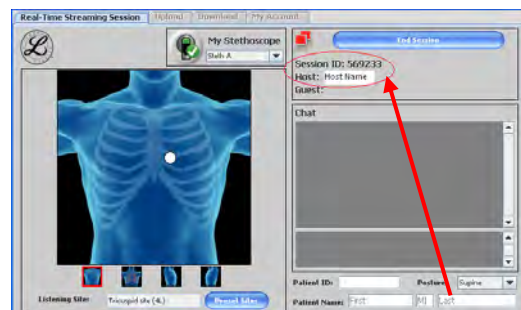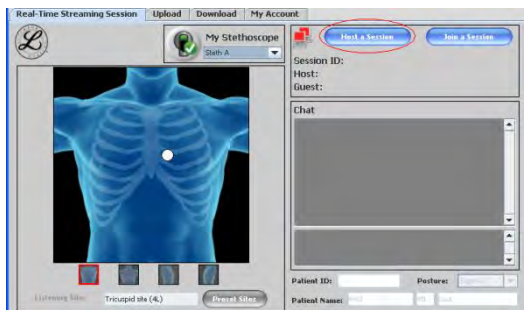

7. The Patient site will always Host a Session; a random session ID number will be generated. Share the Session ID number with the Consultant site.
8. Once successfully connected, the computer icons will turn green. Follow the direction of the consultant regarding requested placement.



9. When patient assessment is complete, click on End Session and log out of the software.
10. Disconnect by right clicking the AnyConnect icon (🔒) in the taskbar and select Disconnect from the pop-up menu.